


The Med X Change EvolutionHD digital recorder software version 3.x requires actions to be taken prior to applying the WannaCry vulnerability patch. Follow the steps below to apply the patch.


1. Download the vulnerability patch and copy it to a removable USB drive.
2. Connect a USB keyboard and USB mouse to available USB ports.
3. Power on the EvolutionHD and allow it to boot to the *Main* screen.
4. At any time, type medx123 on the keyboard to gain access to the operating system environment.
5. From the *Utilities* folder, type E:\ in the address bar, then press *Enter* to open the attached USB drive containing the vulnerability patch.

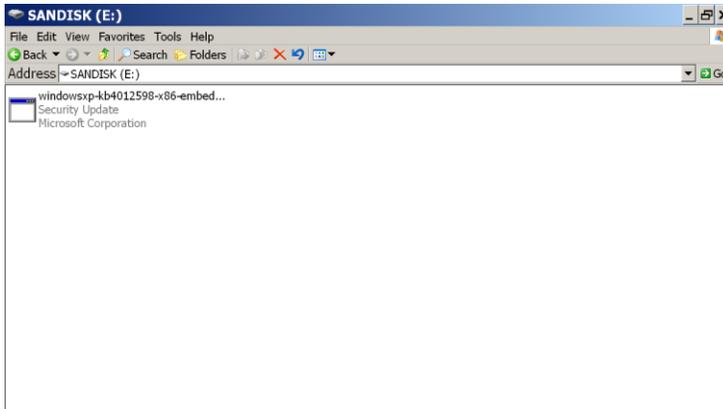

6. Double-click the patch on the desktop to begin applying the patch and follow the instructions on the screen.
7. When the patch finishes installing, click *Finish*.



8. Type *medx123* again to load the Utilities folder.
9. Double-click the Restart icon to complete the patch procedure.

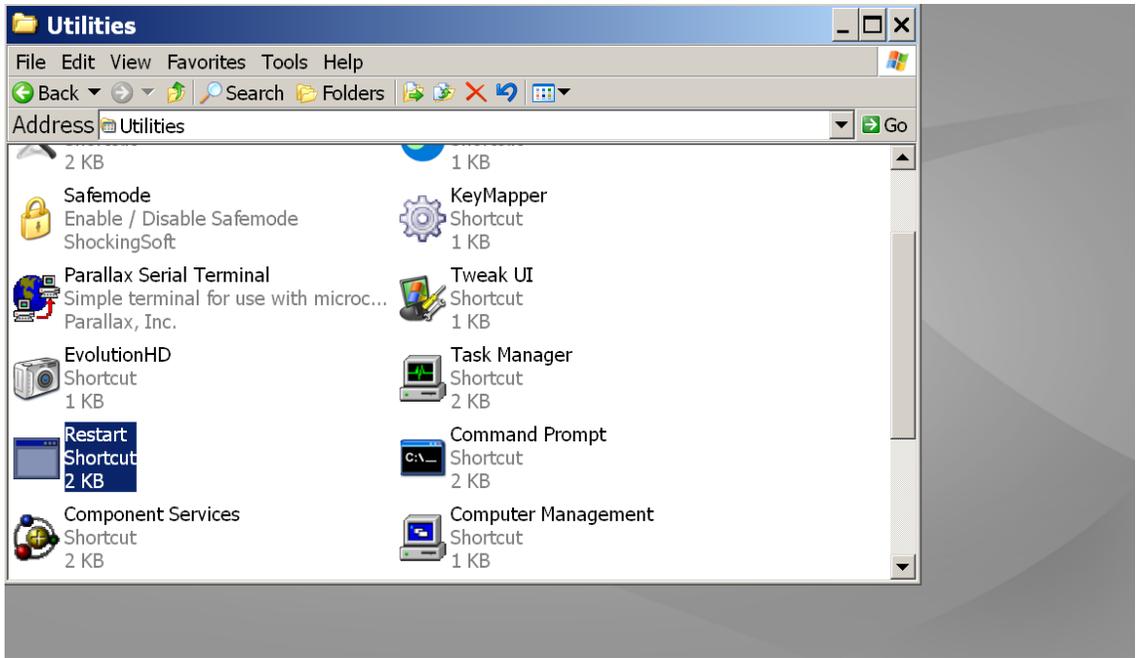

10.