


# QuickTime Pro

## How to import, make a simple edit, and save a video file.

\* Convert the .wmv video format to .mov format for editing with Apple based video editing applications.

- 1) Click the QuickTime Player icon in the dock. If not there, open the Applications folder and double-click on the QuickTime Player icon. Import the video by selecting File > Open File... Select the video file from the proper directory and click open.

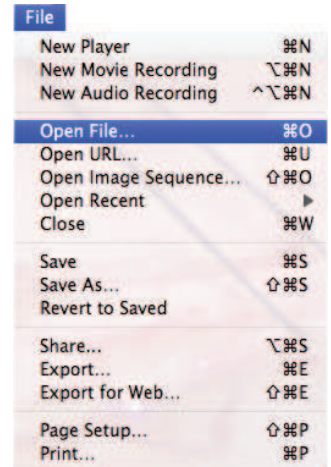


- 2) Drag the playhead to where you want your scene to begin. Move the crop markers to select the footage that you want to keep. The shaded portion highlights the footage you have selected. Choose Edit > Trim to Selection to keep the portion of video and remove the rest.



Slide the in and out pointer to select a portion of video.



- 3) When you are done editing; Click File>Save As... In the 'Save As:' box give your clip a name and select where you would like the movie saved. Tick the radio button 'Save as a self-contained movie'. Then click 'Save'.

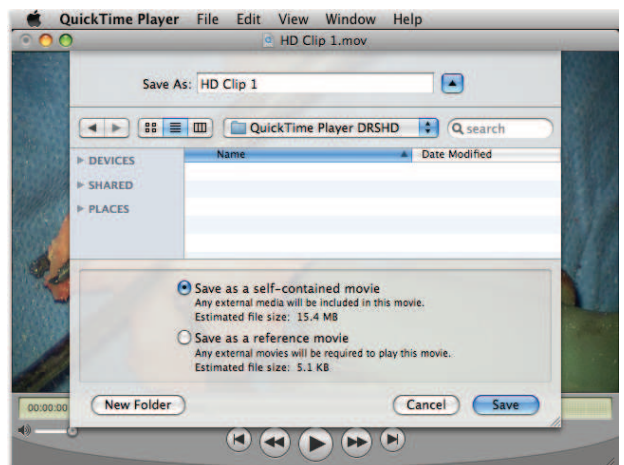

Toll-Free 1-888-982-4469  
www.medxchange.com