

WINDOWS MOVIE MAKER


How to import, make a simple edit, and save a video file.

To optimize the quality of your edited clip, please contact Med X Change to receive a custom profile for Windows Movie Maker to load onto your computer. Once you load this file you can proceed to the steps below.



- 1) Click "Import Media" or File>Import Media. Choose your video and click "Import".



- 2) Click on the video segment and now you can make your edits (mark-in and mark-out etc...)

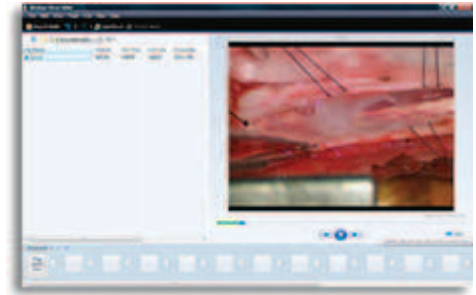

- 3) Once done, click your edited clip and drag to the Storyboard below.

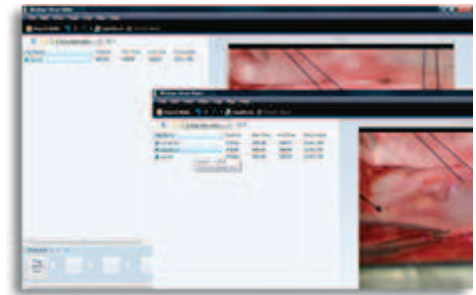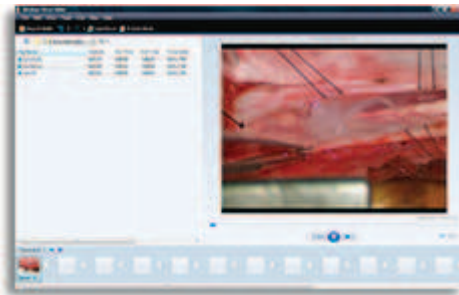

- 4) Click "Publish Movie" or File>Publish Movie.



Toll-Free 1-888-982-4469
www.medxchange.com

- 5) From here, select the name and location on your computer to save the edited clip.



- 6) The next screen that will pop-up will be the settings. From here, scroll down and select the "HDMD" Profile that we earlier imported and click "Publish".

